

Modelling resin, Autopolymer

Instructions for use

1. Areas of application:

Manufacture / Modelling of

- ☐ telescopic and conus crowns
- ☐ individual attachments
- ☐ fixations / temporary splints for soldering
- ☐ inlays / onlays / connectors and clasps
- ☐ Maryland bridges
- ☐ direct intraoral core build-ups
- ☐ implant superstructures

2. Contraindications:

In the event of a proven allergy to one or more ingredients of the product, the product must not be used. If the material comes in contact with oral mucosa on direct application, rinse the mouth thoroughly with water.

3. Instructions for use - Telescopic- and Conus crowns:

Preparation: Place the corresponding quantity of fluid and powder in a separate mixing beaker.

Processing: Dip the tip of the brush into the liquid and then pick up a small amount of powder. The resin swells on the brush tip and can be applied exactly onto the model. Repeat this process several times and thereby build up the secondary crown progressively. Curing time: approx. 3 minutes (depending on ambient temperature).

After completion of the secondary crown, carefully remove it and check the margin and the inside surface. Reposition the secondary crown onto the primary crown and grind carefully to a uniform thickness of approximately 0,3 to 0,4mm using an appropriate bur.

Embedding, burning out and casting are carried out in the same way as the waxing technique.

4. Advice:

- ☐ If a plaster model is being worked on, isolation should be used as separating agent
- ☐ Clean the brush after use or, if necessary, during the work procedure with the liquid
- ☐ Do not return leftover liquid or powder to the bottle

5. Storage and Safety Measures:

Do not use the material after the expiration date. Store at temperatures not exceeding 25°C [77°F]. Protect from sunlight. Close containers immediately after use. Keep away from heat/sparks/open flames/hot surfaces. No smoking. Dispose of contents/container in accordance with local disposal regulations.

Highly flammable liquid and vapour. Causes skin irritation. May cause an allergic skin reaction. May cause respiratory irritation. Avoid breathing dust/fume/gas/mist/vapours/spray. Wear protective gloves/protective clothing/eye protection/ face protection. IF ON SKIN: Wash with plenty of soap and water.


6. Delivery forms:

Color: red

100g, 500g, Powder

100ml, 500ml Liquid

1 Set: 100g Powder and 80ml Liquid with accessories

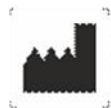

Via Torrette Battifoglia Zona Industriale, Fraz. S.Andrea Delle Fratte, 06132 Perugia Italy P.Iva 03533540542

info@micropolymerdental.com

www.micropolymerdental.com

CE 0425
Class IIa

Phone: (+39) 327 4572 561 

Phone/Fax (+39) 0924 49 089 