

microAcry Bond Istruzioni d'uso

Primer Resina/denti Resina/Resina

microAcry Bond it is a primer for conditioning acrylic surfaces in particular

for the adhesion of the teeth to the resin for mobile prostheses, after pressing the injection or casting.

microAcry Bond it is composed of alcohol-based methyl methacrylate free of glycerin unlike the classic monomer liquids of the abase methyl methacrylate resins

After application thanks to the alcohol base and the absence of glycerin, it conditions the surface and evaporates quickly, without greasing the surface of the resin or splint teeth, ensuring perfect chemical retention.

Technique of use:

Before application, we recommend making retentions with a resin cutter or sandblasting with aluminum oxide.

Apply with a soft brush to wait 2-3 minutes for evaporation

microAcry Bond guarantees a chemical adhesion between splint teeth and acrylic resins, hot and cold thermopolymerizing

Packaging: **30 ml** bottle

Phone: (+39) 327 4572 561  Phone/Fax +39 0924 49 089